

QUICK START GUIDE

Get started with LinX quickly and easily with this Quick Start Guide.

READ THIS INSERT AND ALL OF THE LABELLING PROVIDED WITH THE CGM APP BEFORE HANDLING THE SENSOR KIT.

Product name: Continuous Glucose Monitoring System Sensor
Trade Name: LinX CGMS
Product Model: GX-01, GX-02, GX-01S, GX-02S
For use with: RC2107, RC2109 CGM app

STEP 1: Install LinX CGM App

Search LinX CGM on the google play or App store.



STEP 2: Register an Account


Use an e-mail account to register a LinX account.

STEP 3: Apply the Sensor

Description:

The Sensor is located inside the Sensor Applicator. Follow the instructions to prepare and apply the Sensor on the back of your upper arm (age ≥18 years), or on the abdomen (age 2-17 years). The Sensor has a small, flexible tip that is inserted just under the skin. The Sensor can be worn for up to 15 days.

- Action 1** Open the package and select Insertion Area
Upper arm: The back of the upper arm (Don't insert into the muscles on the outer side of the upper arm.)
Abdomen: Avoid inserting around the navel, scars, subcutaneous induration, and the waistband.
- Action 2** Disinfect the insertion site with an alcohol wipe and let it dry completely.
- Action 3** Unscrew the cover of the applicator and set it aside.
- Action 4** Align the opening of the applicator with the insertion site tightly and press the white insertion button of the applicator. After hearing the sound of the spring retracting, the sensor will be inserted underneath the skin and the insertion needle will automatically retract back into the applicator.
- Action 5** Gently pull the sensor applicator away from the body, and the sensor should now be attached to the skin.
- Action 6** After installing the sensor, make sure that the sensor is firmly in place. Use your fingers to flatten the edge of the sensor to avoid wrinkles and warping.

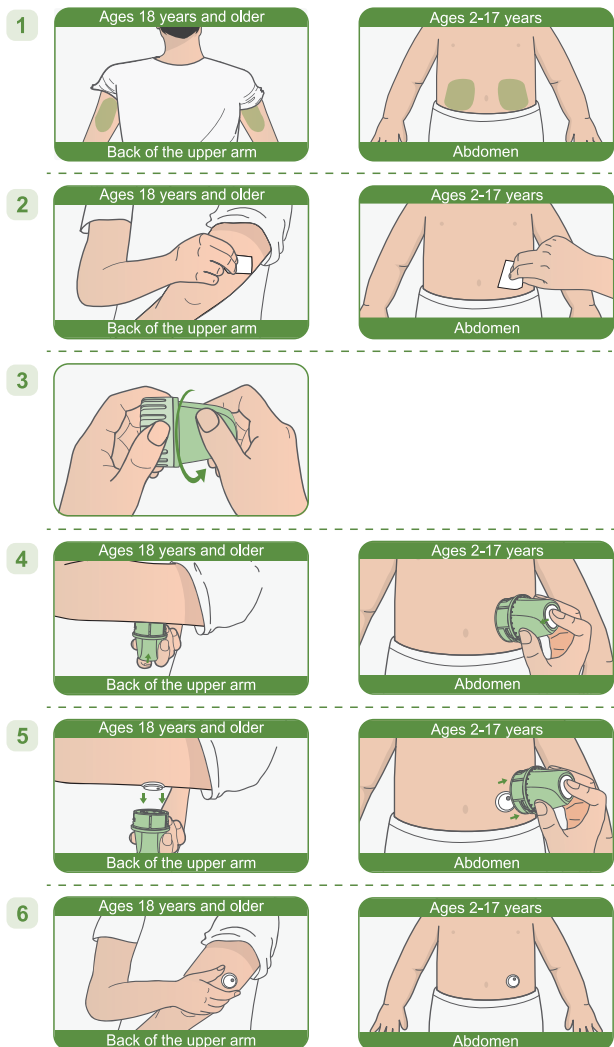

STEP 4: Pair sensor with LinX CGM App

- (1) Log in on the LinX CGM app.
- (2) Make sure the phone's Bluetooth is switched on.
- (3) Click "pairing" on the home dashboard.
- (4) Select and click the correct SN number of the sensor listed on the Nearby Device list, (Make sure the selected SN number is in accordance with the number printed on the packaging box and the applicator cover.)
- (5) Wait for a successful pairing.

Refer to the User Guide for more details of pairing procedures and the subsequent use.

The electronic manual should be accessible for viewing using a smartphone or computer with a web browser and PDF viewing capability. If you choose to view the electronic manual on the LinX CGM App, make sure to download the application on the compatible device models listed in our equipment list.

Please refer to the following website to check the compatible device list and User Guide:

<https://www.microtechmd.com/support/download/664/666>

Contraindications:

The Continuous Glucose Monitoring System must be removed prior to Magnetic Resonance Imaging (MRI).

Don't wear your CGM sensor for computed tomography (CT) scan, or high-frequency electrical heat (diathermy) treatment.

Taking higher than the maximum dose of acetaminophen (e.g. > 1 gram every 6 hours in adults) may affect the CGMS readings and make them look higher than they really are.


The CGM System was not evaluated for the following persons:

- Pregnant women
- Peritoneal dialysis patients
- Patients with implanted pacemakers
- Patients with coagulation disorders or those taking anticoagulant drugs.

Precautions:

- When children and adolescents use the CGM system, their legal guardians, caregivers or healthcare professionals shall operate the CGM system on their behalf.
- No modifications to the Continuous Glucose Monitoring System Sensor are allowed. Unauthorized modification of the CGMS may cause the product to malfunction and become unusable.
- Before using this product, you need to read the Instruction Manual or be trained by a professional. No doctor's prescription is required for use at home.
- The CGMS contains many small parts that can be dangerous if swallowed.
- During rapid changes in blood glucose (more than 0.1 mmol/L per minute) e.g., during exercise or after taking hypoglycemic drugs, glucose levels measured in interstitial fluid by the CGMS may not be the same as blood glucose levels. When blood glucose levels drop rapidly, the sensor may produce a higher reading than the blood glucose level; Conversely, when blood glucose levels rise rapidly, the sensor may produce a lower reading than the blood glucose level. In these cases, the sensor's reading should be checked by a fingertip blood test using a glucose meter.
- Severe dehydration or excessive loss of water may result in inaccurate results. When you suspect you are dehydrated, consult a health care professional immediately.
- If you think the CGMS sensor reading is inaccurate or inconsistent with the symptoms, use a blood glucose meter to test your blood glucose level or calibrate the glucose sensor. If the problem persists, remove and replace the sensor.
- The performance of the CGMS has not been evaluated when used with another implantable medical device, such as a pacemaker.
- Details of what interferences may affect the accuracy of the detection are given in "Potential Interference information".
- If the sensor loosens or is taken off, it may result in no readings displayed on the App.
- If a sensor tip breaks, do not handle it yourself. Please seek professional medical help.
- This product is water resistant and can be worn during showers and swimming, but do not bring sensors into water more than 2 meters deep for longer than 1 hour.
- While extensive user testing was done on LinX CGMS in Type 1 and Type 2 diabetic patients, the study groups did not include women with gestational diabetes.
- If the product is not working properly or has been damaged, stop using the product.
- Maintain a distance of at least 2–3 inches (5–7.5 cm) between insulin injection sites and the CGM sensor to minimize interference.

Symbols:

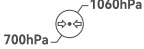Refer to Instruction Manual	
Atmospheric pressure limitation	
Type BF Applied Part	
Single sterile barrier system with protective packaging outside using irradiation	
Manufacturer	
Do not use if package is damaged and consult instructions for use	
Batch code	
Importer	
MR unsafe	
Authorised Representative in the European Union	
Temperature limit	
Humidity limitation	
Do not re-use	
Caution	
Date of manufacture	
Use-by date	
Serial number	
Unique device identifier	
Waste Electrical and Electronic Equipment (WEEE) – Follow local requirements for proper disposal	
CE Mark	
Medical device	
Instructions for use	
The level of protection against ingress of solid foreign objects is 6 (Protected against access to hazardous parts with a wire) The level of protection against ingress of water with harmful effects is 8 (Protected against the effects of continuous immersion in water)	IP68



MicroTech Medical (Hangzhou) Co., Ltd.
No.108 Liuze St., Cangqian, Yuhang District,
Hangzhou, 311121 Zhejiang, P.R.China

 **CMC Medical Devices & Drugs S.L**
C/Horacio Lengo N° 18 CP 29006,
Málaga-Spain

Authorised distributor:
Homemed (Pty) Ltd
Co Reg 2003/022932/07
100 Sovereign Dr, Route 21 Corporate Park,
Nellmapius Dr, Irene, 0157, South Africa
Tel: 0861 106 150 Helpline: 079 832 6825
E: diabetes@homemed.co.za
W: www.homemed.co.za ; www.linxcgm.co.za





Quick Start Guide

QUICK START GUIDE

Get started with LinX quickly and easily with this Quick Start Guide.

READ THIS INSERT AND ALL OF THE LABELLING PROVIDED WITH THE CGM APP BEFORE HANDLING THE SENSOR KIT.

Product name: Continuous Glucose Monitoring System Sensor
Trade Name: LinX CGMS
Product Model: GX-01, GX-02, GX-01S, GX-02S
For use with: RC2107, RC2109 CGM app

STEP 1: Install LinX CGM App

Search LinX CGM on the google play or App store.

SEARCH **LinX CGM**



STEP 3: Apply the Sensor

Description:

The Sensor is located inside the Sensor Applicator. Follow the instructions to prepare and apply the Sensor on the back of your upper arm (age ≥18 years), or on the abdomen (age 2-17 years). The Sensor has a small, flexible tip that is inserted just under the skin. The Sensor can be worn for up to 15 days.

- Action 1** Open the package and select Insertion Area
Upper arm: The back of the upper arm (Don't insert into the muscles on the outer side of the upper arm.)
Abdomen: Avoid inserting around the navel, scars, subcutaneous induration, and the waistband.
- Action 2** Disinfect the insertion site with an alcohol wipe and let it dry completely.
- Action 3** Unscrew the cover of the applicator and set it aside.
- Action 4** Align the opening of the applicator with the insertion site tightly and press the white insertion button of the applicator. After hearing the sound of the spring retracting, the sensor will be inserted underneath the skin and the insertion needle will automatically retract back into the applicator.
- Action 5** Gently pull the sensor applicator away from the body, and the sensor should now be attached to the skin.
- Action 6** After installing the sensor, make sure that the sensor is firmly in place. Use your fingers to flatten the edge of the sensor to avoid wrinkles and warping.


Indication for use:

The Continuous Glucose Monitoring System sensor is a real time, continuous glucose monitoring device. When the system is used together with compatible devices, it is indicated for the management of diabetes in persons 2 years and older. It is designed to replace fingerstick blood glucose testing for diabetes treatment decisions. Interpretation of the system results should be based on the glucose trends and several sequential readings over time. The system also detects trends and tracks patterns, and aids in the detection of episodes of hyperglycemia and hypoglycemia, facilitating both acute and long-term therapy adjustment.

STEP 2: Register an Account

Use an e-mail account to register a LinX account.

Email

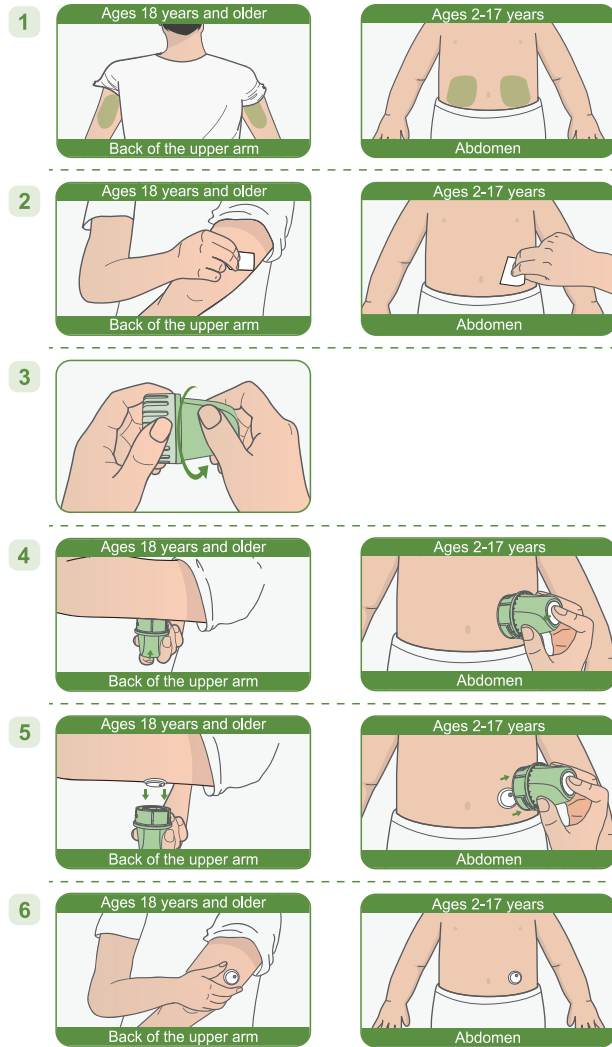

STEP 4: Pair sensor with LinX CGM App

- (1) Log in on the LinX CGM app.
- (2) Make sure the phone's Bluetooth is switched on.
- (3) Click "pairing" on the home dashboard.
- (4) Select and click the correct SN number of the sensor listed on the Nearby Device list. (Make sure the selected SN number is in accordance with the number printed on the packaging box and the applicator cover.)
- (5) Wait for a successful pairing.

Refer to the User Guide for more details of pairing procedures and the subsequent use.
The electronic manual should be accessible for viewing using a smartphone or computer with a web browser and PDF viewing capability. If you choose to view the electronic manual on the LinX CGM App, make sure to download the application on the compatible device models listed in our equipment list.

Please refer to the following website to check the compatible device list and User Guide:

<https://www.microtechmd.com/support/download/664/666>



Quick Start Guide

Contraindications:

- Don't wear your CGM sensor for computed tomography (CT) scan, or high-frequency electrical heat (diathermy) treatment. Taking higher than the maximum dose of acetaminophen (e.g. > 1 gram every 6 hours in adults) may affect the CGMS readings and make them look higher than they really are.
- The CGM System was not evaluated for the following persons:
 - Pregnant women
 - Peritoneal dialysis patients
 - Patients with implanted pacemakers
 - Patients with coagulation disorders or those taking anticoagulant drugs.

Precautions:

- When children and adolescents use the CGM system, their legal guardians, caregivers or healthcare professionals shall operate the CGM system on their behalf.
- No modifications to the Continuous Glucose Monitoring System Sensor are allowed. Unauthorized modification of the CGMS may cause the product to malfunction and become unusable.
- Before using this product, you need to read the Instruction Manual or be trained by a professional. No doctor's prescription is required for use at home.
- The CGMS contains many small parts that can be dangerous if swallowed.
- During rapid changes in blood glucose (more than 0.1 mmol/L per minute) e.g., during exercise or after taking hypoglycemic drugs, glucose levels measured in interstitial fluid by the CGMS may not be the same as blood glucose levels. When blood glucose levels drop rapidly, the sensor may produce a higher reading than the blood glucose level; Conversely, when blood glucose levels rise rapidly, the sensor may produce a lower reading than the blood glucose level. In these cases, the sensor's reading should be checked by a fingertip blood test using a glucose meter.
- Severe dehydration or excessive loss of water may result in inaccurate results. When you suspect you are dehydrated, consult a health care professional immediately.
- If you think the CGMS sensor reading is inaccurate or inconsistent with the symptoms, use a blood glucose meter to test your blood glucose level or calibrate the glucose sensor. If the problem persists, remove and replace the sensor.
- The performance of the CGMS has not been evaluated when used with another implantable medical device, such as a pacemaker.
- Details of what interferences may affect the accuracy of the detection are given in "Potential Interference information".
- If the sensor loosens or is taken off, it may result in no readings displayed on the App.
- If a sensor tip breaks, do not handle it yourself. Please seek professional medical help.
- This product is water resistant and can be worn during showers and swimming, but do not bring sensors into water more than 2 meters deep for longer than 1 hour.
- While extensive user testing was done on LinX CGMS in Type 1 and Type 2 diabetic patients, the study groups did not include women with gestational diabetes.
- If the product is not working properly or has been damaged, stop using the product.
- Maintain a distance of at least 2–3 inches (5–7.5 cm) between insulin injection sites and the CGM sensor to minimize interference.

Symbols:

Refer to Instruction Manual	
Atmospheric pressure limitation	
Type BF Applied Part	
Single sterile barrier system with protective packaging outside using irradiation	
Manufacturer	
Do not use if package is damaged and consult instructions for use	
Batch code	
Importer	
MR unsafe	
Authorised Representative in the European Union	
Temperature limit	
Humidity limitation	
Do not re-use	
Caution	
Date of manufacture	
Use-by date	
Serial number	
Unique device identifier	
Waste Electrical and Electronic Equipment (WEEE) – Follow local requirements for proper disposal	
CE Mark	
Medical device	
Instructions for use	
The level of protection against ingress of solid foreign objects is 6 (Protected against access to hazardous parts with a wire) The level of protection against ingress of water with harmful effects is 8 (Protected against the effects of continuous immersion in water)	



MicroTech Medical (Hangzhou) Co., Ltd.
No.108 Luze St., Cangqian, Yuhang District,
Hangzhou, 311121 Zhejiang, P.R.China

EU REP CMC Medical Devices & Drugs S.L.
C/Horacio Lengo Nº 18 CP 29006,
Málaga-Spain

Authorised distributor:
Homemed (Pty) Ltd
Co Reg 2003/022932/07
100 Sovereign Dr, Route 21 Corporate Park,
Nellmapius Dr, Irene, 0157, South Africa
Tel: 0861 106 150 Helpline: 079 832 6825
E: diabetes@homemed.co.za
W: www.homemed.co.za ; www.linmcgm.co.za


CE 0197




1034-PMTL-569,V03
Effective date: 2026-04-01

折叠方式


外折线 ————
内折线 - - - - -

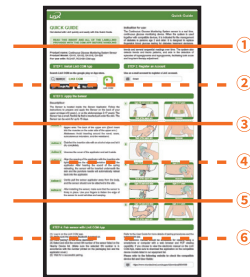

纵向7等分

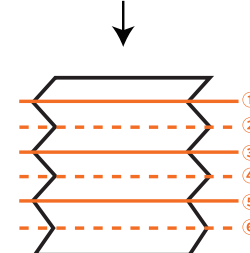

横向2等分

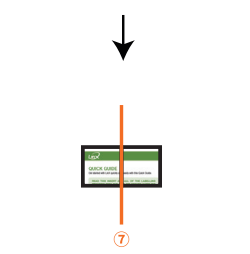

横向2等分



折叠后45*45mm, 正面logo朝上
(尽量压扁, 翘起角度不宜过高)



本设计图纸版权归本公司所有, 未经本公司书面同意不得复印。

本图纸应与说明, 产品状况, 产品结构图及有
关图纸协调使用, 发现任何差异请立即通知设计师。

图档中所有尺寸比例为1:1.
公差: L(长)*(宽): +1/-1mm,H(高): ±1mm

项目信息

名称 LinX 快操卡(南非版 2+)

图纸号
物料号 1034-PMTL-569
版本号 V03

版本变更描述
1.product name下方新增一行:
Trade Name: LinX CGMS
2.气压符号单位更正为hPa

发放范围 生产 () 采购 (√) 计划 (√) 其他:

切刀 ————

压线 ————

开槽 ————

半穿 ————

粘合

备注
无印刷

尺寸: 180*315mm, 折叠后45*45mm

印刷颜色: 四色印刷

材质: 80g双胶纸

特种工艺:

备注: 折叠, 见图纸

设计审核

设计

审核

批准

生效日期

日期